Hippocampe Wheelchair First Use and Precautions for Use

First Use:

- 1. As is the case with classic wheelchairs, there is a period of adaptation for using the Hippocampe.
- 2. The exterior all terrain wheels serve as a hand rim, for self propelling users, the grip inside the rim provides better handling.
- 3. The balloon wheels required the assistance of a carer/personal assistant.

Precautions for Use:

- 1. The Hippocampe wheelchair **is not a boat**. It allows entry into the water **up to the height of the wheelchair axle** and is not designed to float with the user in it.
- 2. Do not immerse the Hippocampe wheelchair water must not go above the height of the rear axle.
- 3. The Hippocampe wheelchair is designed for a maximum weight of 130kg = 20.47 stone.
- 4. Wearing a buoyancy aid is obligatory if entering the water. It is also obligatory to remain within the designated swimming area noted by signs located on the beach.
- 5. On entering water, it is recommended that you lean forward. (The buoyancy of the front wheel can throw you off balance backwards).
- 6. A carer/personal assistant is required for entering the water.
- 7. Users must be pushed/assisted exclusively by capable adults (especially when balloon wheels in use).
- 8. The utmost caution is recommended on beaches and steep slipways (apply breaks).
- 9. Do not use the Hippocampe wheelchair when a wheel is punctured or flat.
- 10. Using the Hippocampe wheelchair with more than one person in the seat **is prohibited**.
- 11. Standing up on the seat or the footrest guard **is prohibited**.
- 12. Modifying the structure of the wheelchair **is prohibited**.
- 13. Avoid going under obstacles with the Hippocampe wheelchair.
- 14. Avoid hand-braking on steep surfaces due to the risk of heating and injury.
- 15. Do not move about on inclinations greater than 15%.
- 16. Do not attempt to clear obstacles larger than 10cm.
- 17. Do not smoke while using the wheelchair, the seat is designed not to catch fire (within 15 seconds) on contact with cigarettes or matches, allowing the user time to react.
- 18. You must always be accompanied, for entering water, or when the requirement of a harness is in use.

I have read/listen	ed, understand and agree	to abide by the terms of use of the Hippod	campe wheelchair.
Mandatory			
Singed	Printed	Mobile Phone Number	(required)